




TO OPEN

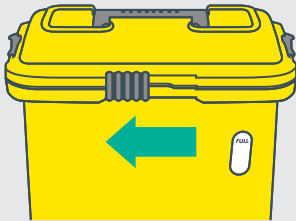

1 Slide the front lock to the left

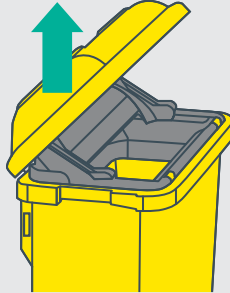

2 Lift lid to 45 degrees for semi-access opening

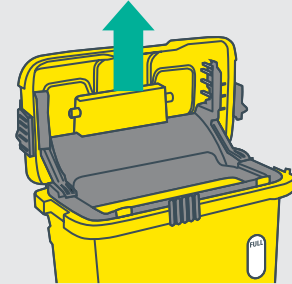


3 Lift lid to 90° for full access opening

USAGE




Drop items pointing downwards into opening

FULL

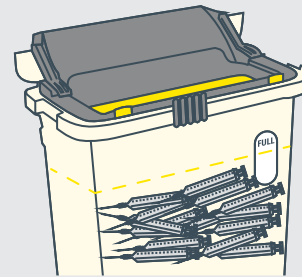


When sharps reach fill level, do not add additional items i.e. overfill collector. The Access Plus collector is full and must be locked and replaced.

CLOSE

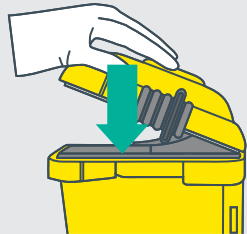


1 Press lid down firmly.

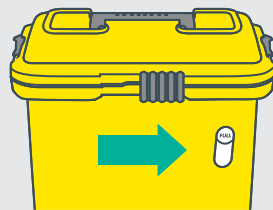

2 Slide the front lock fully to the right.

LOCK & HANDLING

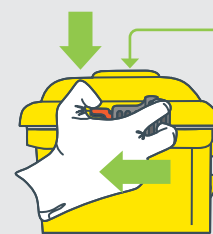

Use handle for manual transport

PLEASE NOTE
Side locks are permanent!

1 Depress red tabs.

2 Slide locks completely forward.

Remember

- The allowable gross mass is 9kg.
- Never put your hands into a sharps collector.
- Always close the collector before moving or removing collector.
- Do not deface or write on the collector.
- Always dispose of sharps and sharps safety devices into sharps collectors.
- The use of an accessory stabilizer is advised.