



OPEN

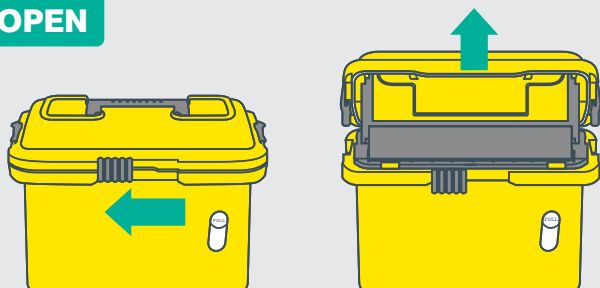


- 1 Slide the front lock to the left.
- 2 Lift lid until it locks open.

SYRINGES

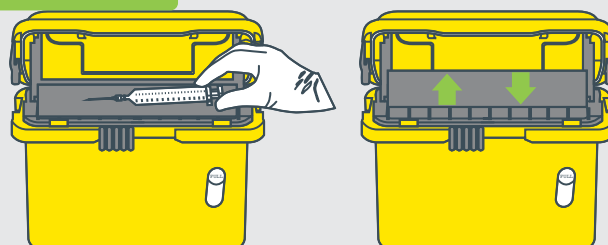


- 1 Drop syringes horizontally onto tray.
- 2 Tray will empty itself.

LIGHT SHARPS

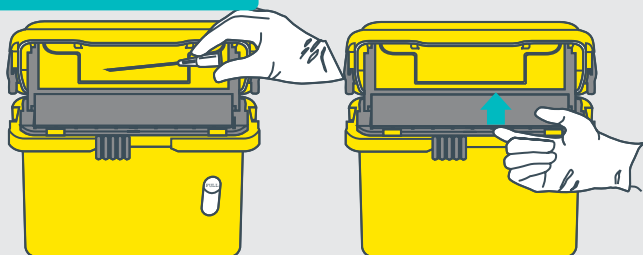


- 1 Drop sharp horizontally onto tray.
- 2 Manually lift the tray to empty.

FULL

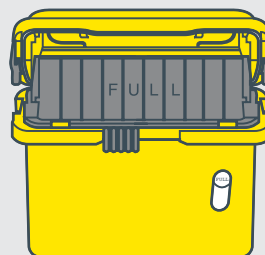

When the tray remains vertical, do not add additional items i.e. overfill collector. The Regular collector is full and must be locked and replaced.

CLOSE

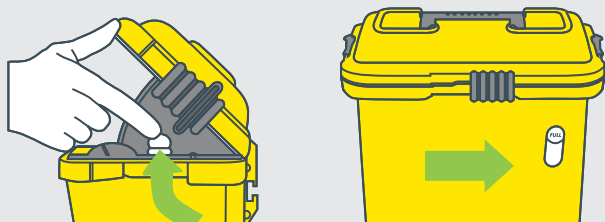

- 1 Press both side supports inward, close lid.
- 2 Slide the front lock fully to the right.

LOCK & HANDLING



Use handle for manual transport

PLEASE NOTE
Side locks are permanent!

- 1 Depress red tabs.
- 2 Slide locks completely forward.

Remember

The allowable gross mass is 9kg.
Never put your hands into a sharps collector.
Always close the collector before moving or removing collector.
Do not deface or write on the collector.
Always dispose of sharps and sharps safety devices into sharps collectors.
The use of an accessory stabilizer is advised.