## **Clostridium Difficile:**

Carriage & Transmission potential by single-use and reusable sharps containers

Authors: Grimmond T, Neelakanta A, Miller B, Saiyed A, Gill P, Cadnum J, Olmsted R, Donskey C, Pate K, Miller K Publication: American Journal of Infection Control 46 (2018) 1154-9 https://doi.org/10.1016/j.ajic.2018.04.206

"...neither DSCs nor RSCs play a role in C. difficile transmission to patients or staff...the assertion that sharps containers may be a fomite for C. difficile transmission is not scientifically feasible." - Authors

#### What was the study?

The study 'A microbiological study to investigate the carriage and transmission potential of Clostridium difficile spores on single-use and reusable sharps containers' adopted a multifaceted epidemiologic, microbiologic, chain-of-infection, and test-of-evidence approach to determine if a relationship between Clostridium difficile infection (CDI) and sharps containers is scientifically feasible.

- Stage 1: 197 Sharpsmart Reusable Sharps

  Containers (RSCs) were sampled for

  C. difficile at processing facilities
- Stage 2: Sharpsmart RSCs were challenged with high C. difficile densities to evaluate efficacy of automated decontamination (Daniels Washsmart process)
- Stage 3: 50 Sharpsmart RSCs and 50
  Disposable Sharps Containers (DSCs)
  were sampled in CDI patient rooms in
  7 hospitals for C. difficile carriage.

#### What were the results?

- **Stage 1:** C. difficile spores were found on 9 of 197 (4.6%) RSCs prior to processing
- **Stage 2:** Decontamination processing completely removed C. difficile
- Stage 3: 4 of 50 RSCs (8%) and 8 of 50
  DSCs (16%) had sub-infective
  counts of
  C. difficile.
- 2 links in the chain of infection were found to be broken
- 5 of 7 tests of evidence for environmental sources of infection were found to be unmet.

# What does this mean for you?

The association of any sharps container with CDI is at odds with chain-of-infection principles, CDI transmission studies, CDI guidelines, the clinical practice of sharps containers being no-touch, and guidelines surrounding sharps container and glove use. In examining epidemiologic, microbiologic, chain-of-infection and test-of-evidence criteria, no scientific evidence or mechanism whereby sharps containers could be implicated in CDI transmission has been found.

The study concludes that neither disposable sharps containers nor reusable sharps containers play a role in C. difficile transmission to patients or staff, and that the assertion that sharps containers may be a fomite for C. difficile transmission is not scientifically feasible.



"Peer review is the critical assessment by experts, of manuscripts submitted to journals... unbiased, independent, critical assessment is an intrinsic part of all scholarly work."

International committee of Medical Journal editors





### The Sharpsmart Difference

Healthcare Waste Management cost efficiencies are derived from the successful deployment of five key elements. At Sharpsmart we work closely with our customers to ensure that all healthcare staff are protected, trained and equipped, that the facilities' risk profile is managed, and waste management costs are minimised.

Safety: Our highest commitment is safety; through intelligent products, systemised solutions and bespoke training, we work with our customers to reduce needlestick

Sustainability: Our reusable products and environmentally responsible treatment

Compliance: Experts in the regulatory complexities governing the healthcare sector, we position ourselves as an astute partner delivering expert guidance and compliant solutions to our customers. All Sharpsmart solutions are vetted against

Education: Our ongoing commitment to educate staff within healthcare ensures